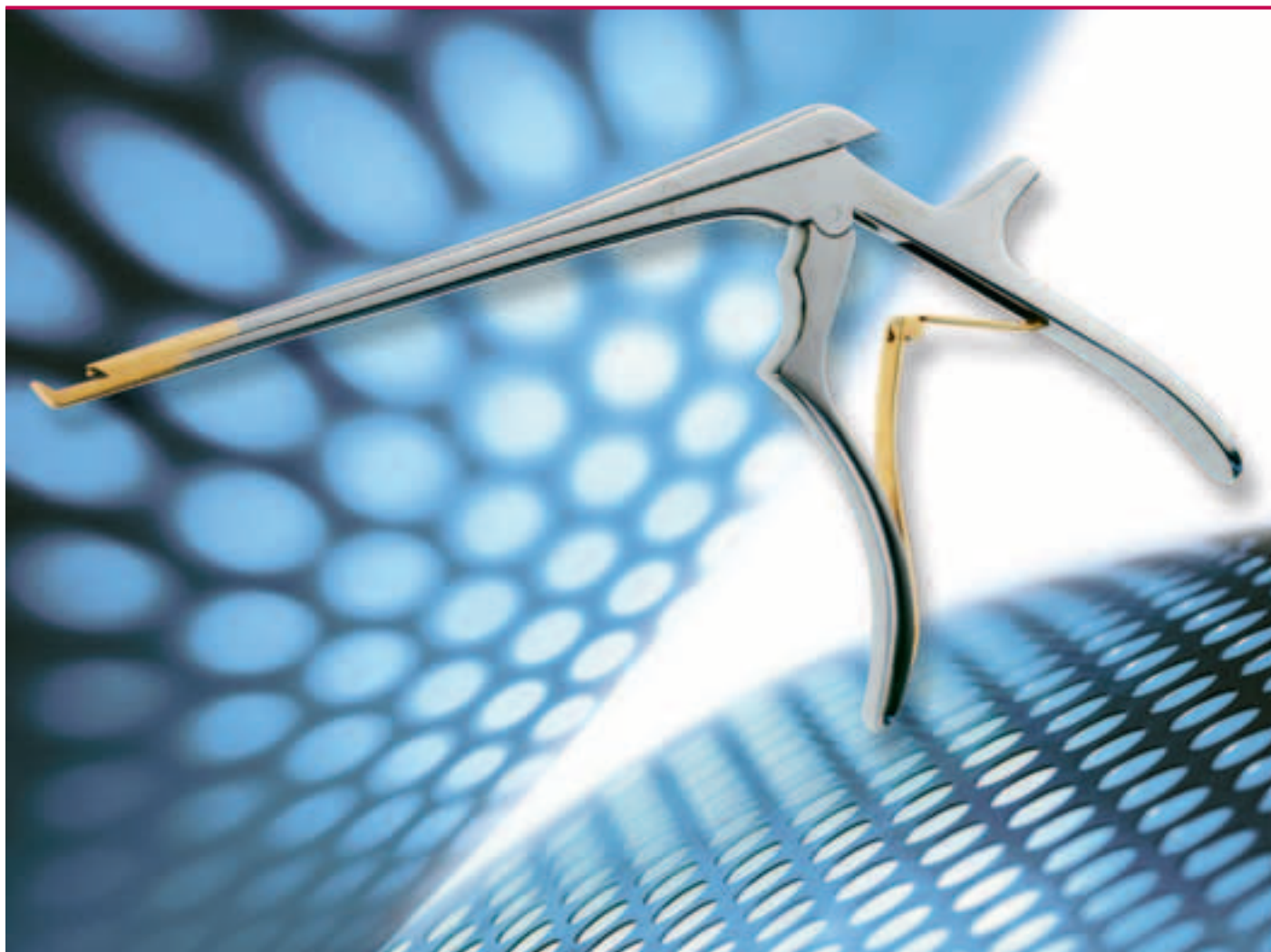
# Aesculap Neurosurgery

## Kerrison Rongeurs

Detachable Bone Punches  
Zerlegbare Knochenstanzen

Detachable in  
Zerlegbar in

1 - 2 - 3

steps  
Schritten

### Disassembly Zerlegen

Remove spring from  
outer part of handle.

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Insert spring into outer  
part of handle.

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Close handle completely  
and pull down the inner  
part of the handle.

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Press up inside part of  
handle and open handle.

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Remove sliding part  
backwards.

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Insert sliding part and  
push it forward.

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## The joy of simplicity

### **Intraoperatively**

Good cutting properties, long durability, easy removal of material.

### **Postoperatively**

Easy disassembly and assembly, efficient cleaning and sterilization.

With the detachable Kerrison bone punches, you bring convenience and reliability to your hospital. The ergonomic handle design guarantees easy handling and fatigue-free working. The bone punch can be disassembled easily and rapidly into two components without using any additional tools. Therefore cleaning and reprocessing is easier and faster.

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
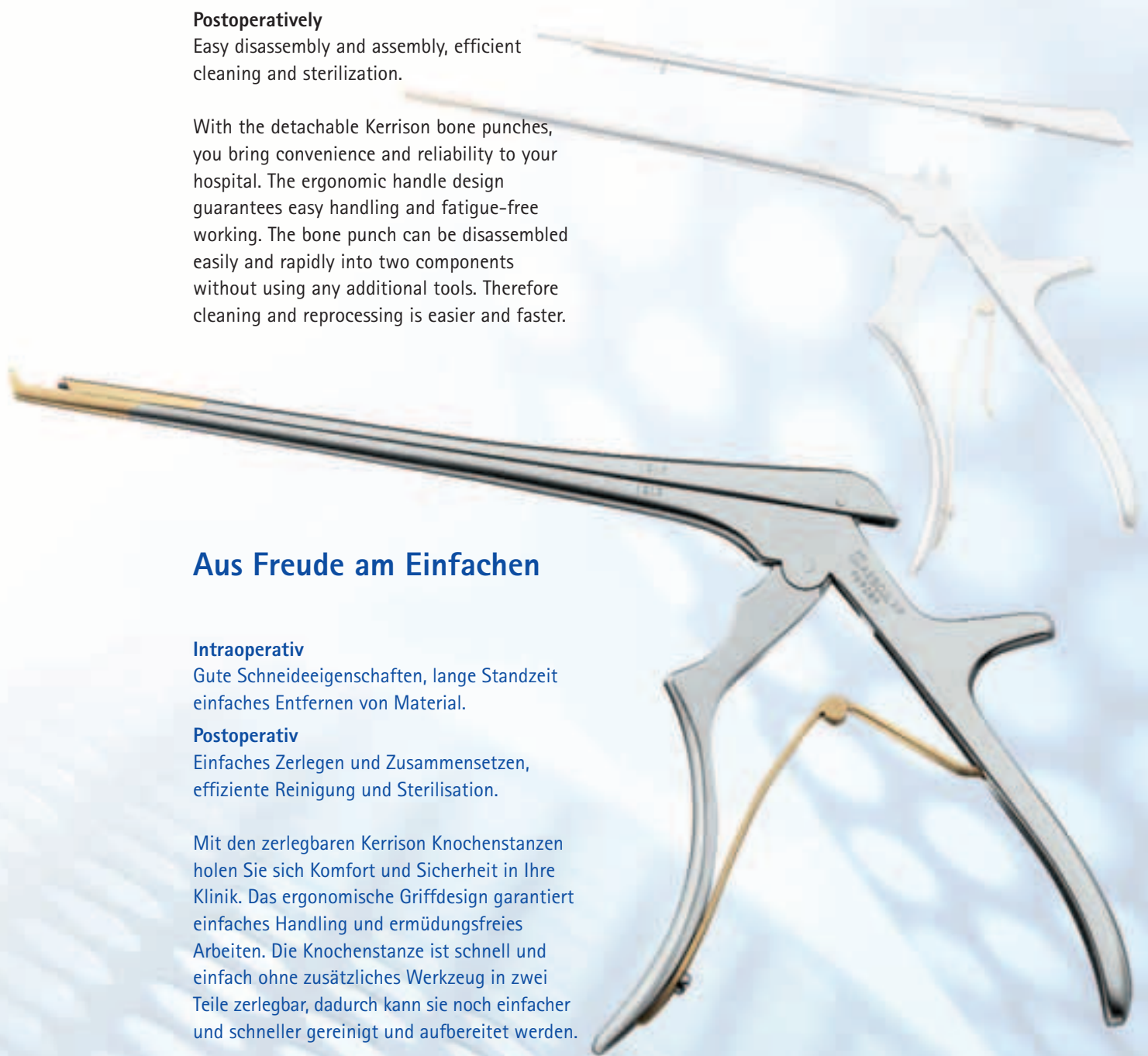
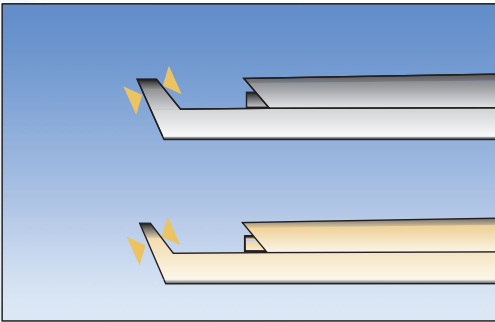
- ▶ Kerrison bone punches with a thin footplate are designed for use on soft tissue or thin bone only. They are identified by a gold colored distal end.

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
- ▶ The unique numerical code enables components to be properly matched for fast and easy reassembly.

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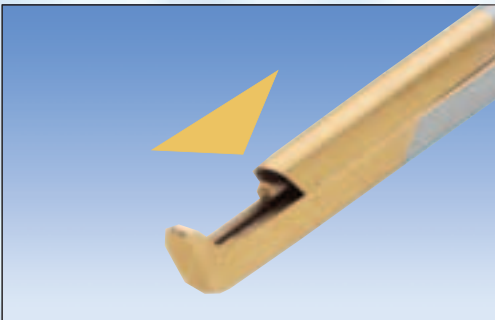- ▶ The material can be easily removed by the "ejector".

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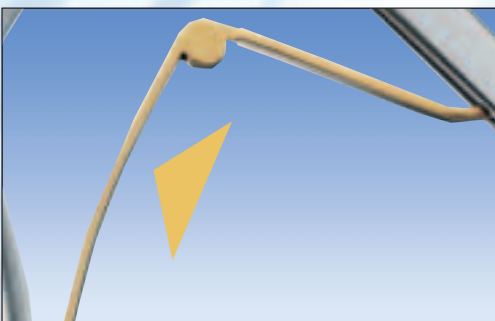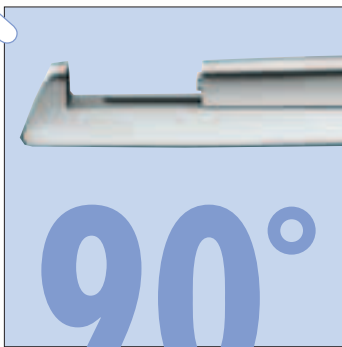
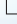
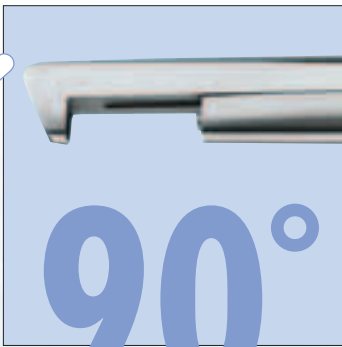
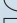
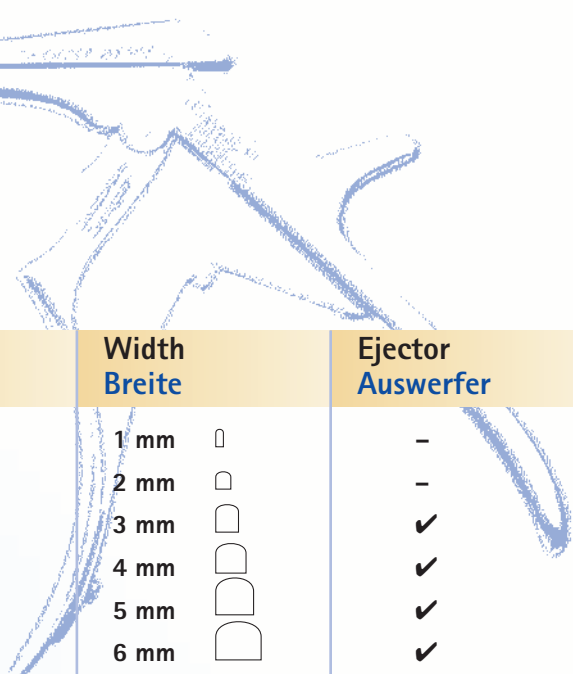
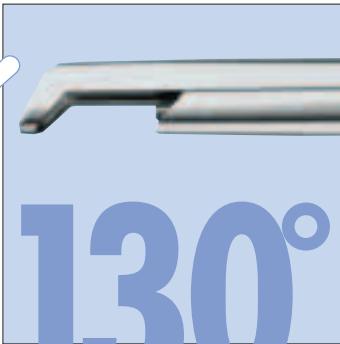
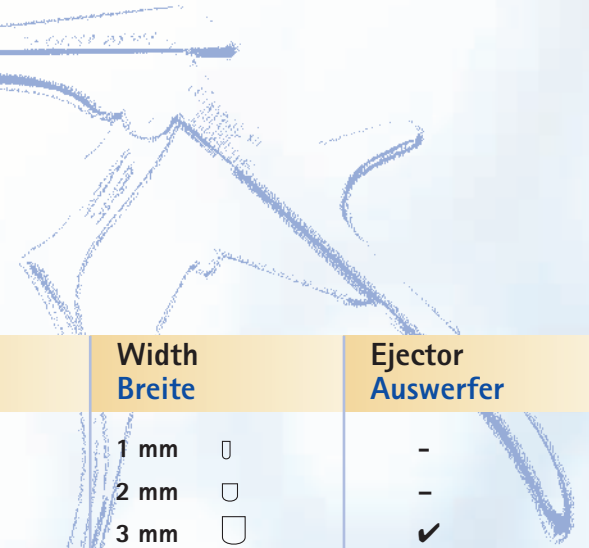
For more information please also ask for our brochures: C085 11 Kerrison Noir, C21802 Kerrison Bone Punches (not detachable).

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Fax +49 7461 95-2600

[www.aesculap.de](http://www.aesculap.de)

All rights reserved. Technical alterations are possible. This leaflet may be used for no other purposes than offering, buying and selling of our products. No part may be copied or reproduced in any form. In the case of misuse we retain the rights to recall our catalogues and pricelists and to take legal actions.

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**Aesculap AG & Co. KG**